


Drivers Install Instructions

Prior to installing the driver make sure that the CPAP is unplugged from the computer


1. You will only need to do these steps, if the boxes do not become colored when plugging in your CPAP
2. Click [HERE](#) to download the drivers
3. Go to your downloads folder and double click on the file

 CDM v2.12.36.4 W....zip

4. Click Extract All

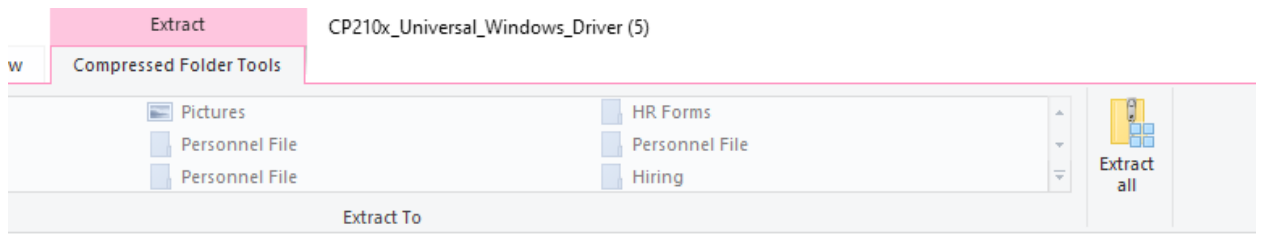


5. Leave the default folder and click extract

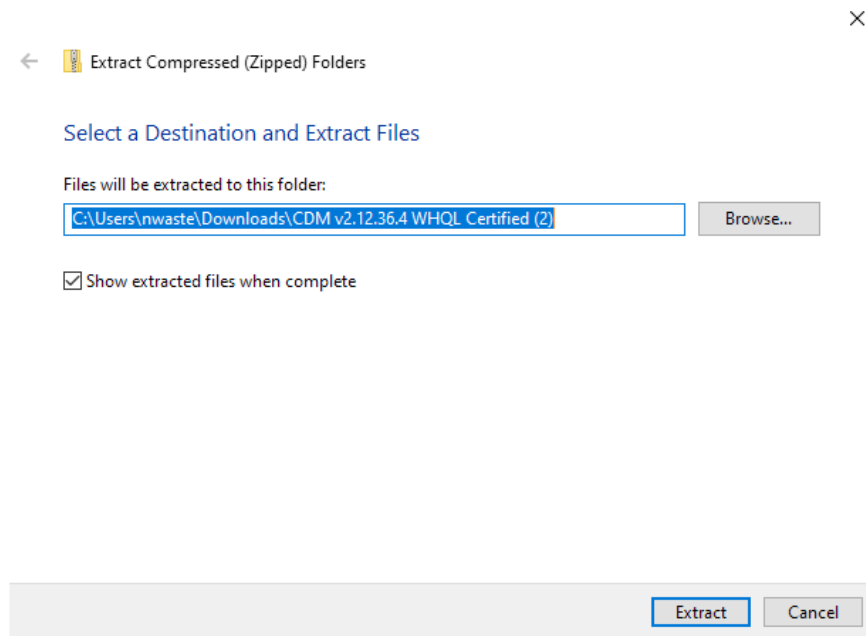


6. Find the file called ftdibus, there are 2 of them. One will have an icon of a piece of paper and gear. Right click on it and choose Install from the menu.




7. You will get a pop up that the operation completed successfully.

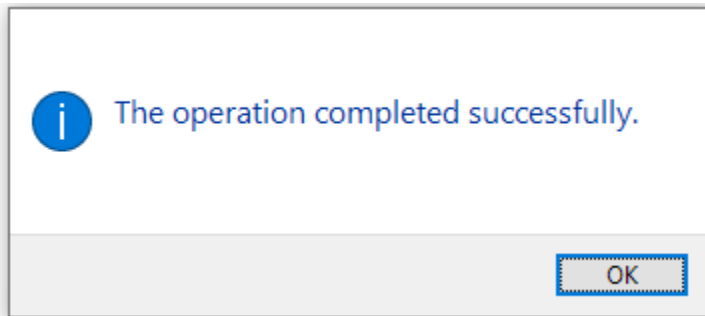

8. Find the file called ftdiport, there are 2 of them. One will have an icon of a piece of paper and gear. Right click on it and choose Install from the menu.

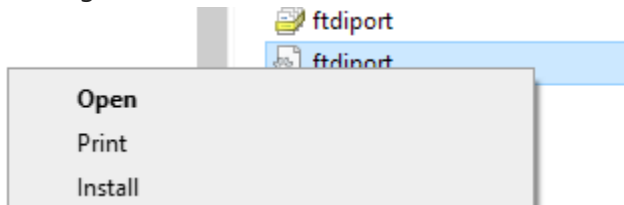

9. You will get a pop up that the operation completed successfully.

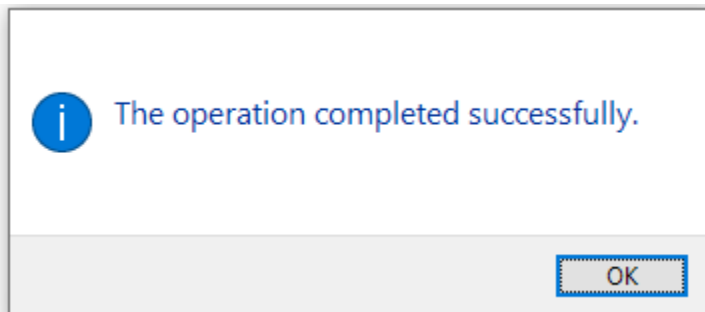

10. Reset the computer, open the MiniCPAP software and plug in the device